**Bilder zur Montageanleitung /**  
pictures to assembly  
instructions

 **BAUERFEIND AG**  
Triebeser Straße 16  
07937 Zeulenroda-Triebes  
Germany  
P +49 (0) 36628 66-40 00  
F +49 (0) 36628 66-44 99  
E info@bauerfeind.com  
**BAUERFEIND.COM**

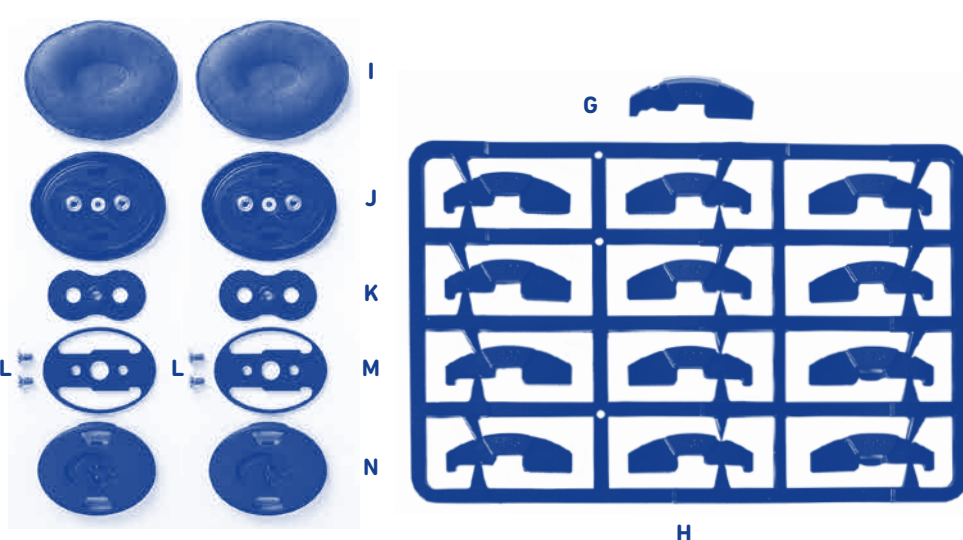
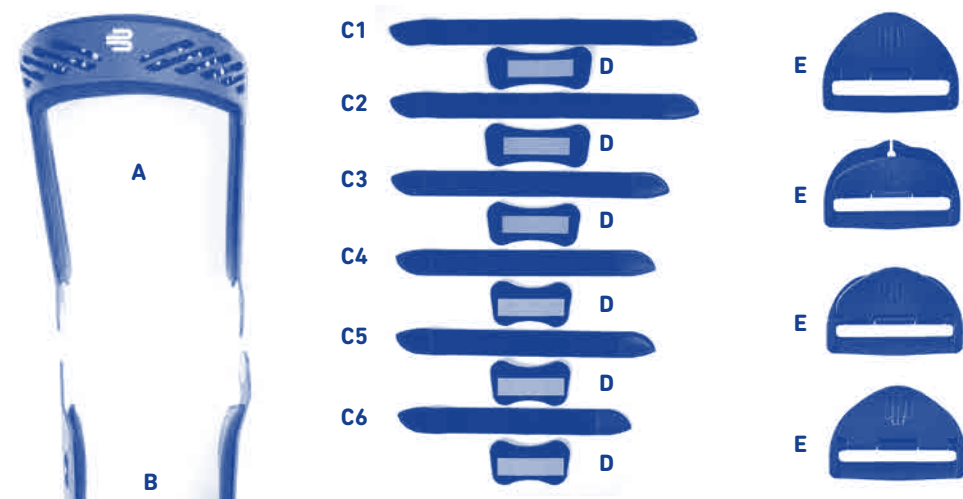
## SecuTec® Genu

Montageanleitung  
Assembly instructions  
Instructions de montage  
Instrucciones de montaje

 **MADE  
IN  
GERMANY**

**Bilder Lieferumfang /**  
pictures to scope  
of delivery

de deutsch.....2	pl polski.....10	ee eesti.....19
en english.....3	cs český.....11	lv latviešu.....19
fr français.....3	sk slovenský.....12	li lietuviai.....20
nl nederlandse.....4	hu magyar.....13	uk українська.....21
it italiano.....5	hr hrvatski.....13	he עברית.....22
es español.....6	sr srpski.....14	ja 日本語.....23
pt português.....7	sl slovenski.....15	ko 한국어.....24
sv svenska.....7	ro romanian.....16	ar عربي.....25
no norsk.....8	el ελληνικά.....16	zh 简体中文.....26
fi suomi.....9	tr türkçe.....17	
da dansk.....10	ru русский.....18	

de deutsch

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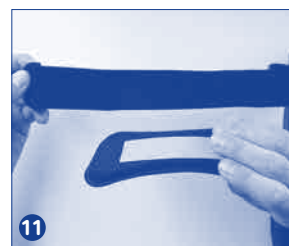
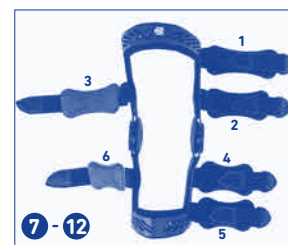
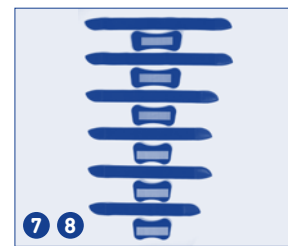
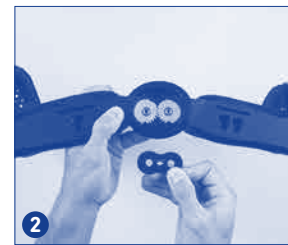
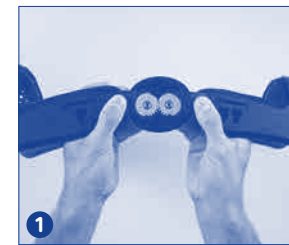
These instructions for use can also be downloaded from:  
[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)

## SecuTec® Genu – Assembly instructions

### Frame components

- A – 1 thigh frame
- B – 1 lower leg frame
- C – 6 straps, sorted by length
- D – 6 cushions, sorted by size
- E – 4 quick-fit fasteners
- F – 2 tibial edge cushions

### Hinge components

- G – 2 limiting wedges, extension 0°
- H – 2 frames, each with 12 limiting wedges
- I – 4 condylar cushions
- J – 2 inner hinge plates, preassembled
- K – 2 outer sliding disks (black)
- L – 4 screws (Torx 20, M 4 × 8)
- M – 2 outer hinge plates
- N – 2 outer hinge covers with screw fastening

### Tool required

Torx 20 screwdriver

**Average assembly time** approx. 5 mins

### Conditions of return

Assembled orthoses may not be returned for credit.

### Possible combinations

The thigh and lower leg frame may differ by a maximum of two sizes.

Size (prox. or distal frame)	1	2	3	4	5	6	7
Size (distal or prox. frame)	1, 2, 3	1, 2, 3, 4	1, 2, 3, 4, 5	2, 3, 4, 5, 6	3, 4, 5, 6	4, 5, 6	5, 6, 7

No product liability will be accepted in the event of failure to comply with the recommended size combination instructions.

### Assembly of the hinges

At each stage of the assembly, ensure that the hinge can move smoothly. If necessary, spray the surfaces of the hinges with Teflon spray.

- 1 Join the thigh frame (A) and lower leg frame (B) together in the extended position, so that the first hinge tooth engages into the first hinge groove on the other side. From the inside, connect this to the **preassembled inner hinge plate (J)** on which the **inner sliding disk** has already been mounted.
- 2 From the outside, position the **outer sliding disk (K)** on the hinge with the notch facing the frame.
- 3 Place the **outer hinge plate (M)** on the outside of the **outer sliding disk (K)** so that the label **Extension** is on the kneecap side and **Flexion** is in the hollow of the knee.
- 4 Fasten the assembled construction elements together using the two Torx 20 **screws (L)**. Tighten the screws hand-tight (3 Nm). Now follow the same procedure (steps 1 to 4) for the hinge on the other side.
- 5 Select the desired flexion and extension wedges from the **frame (H)** or the individually enclosed **limiting wedges (G)** and position them in the hinge (wedge with the label "E" toward **Extension** and "F" toward **Flexion**).
- 6 Click the hinge cover (N) in its open configuration (arrow points to opened lock) onto the hinge from the outside and fix it by rotating the screw fastening through a quarter turn until it points to the symbol with the closed lock.

Now repeat assembly steps 5 and 6 for the hinge on the other side.

### Fitting the straps

- 7 Place the straps lengthwise in front of you. The order of the straps (see photo) reflects their subsequent positions on the orthosis (longest strap at the top, shortest at the bottom). Sort the cushions for the straps in order of size too.
- Turn the orthosis so that the back of it (flexion side) is facing you.
- 8 Start with the longest **strap (C1)**. Thread this strap through the uppermost frame opening (slot) on the thigh frame from the outside to the inside and secure it in place with the Velcro.
  - 9 Repeat this process for the next three straps in turn, threading them into the lateral slots on the thigh / lower leg frame.
  - 10 Now thread the **quick-fit fasteners (E)** onto the exposed ends of the four belts that have just been attached.

### Fitting the cushions

- 11 Secure the **cushions (D)** to the insides of the straps in order of size from proximal to distal. Position the cushions with the shorter long side toward the hinge. Position the cushions which are away from the hinge on the lower leg with the longer long side toward the hinge. Attach the remaining small cushion to the strap on the ventral side of the lower leg frame. Fasten the additional sixth large cushion to the strap on the ventral side of the thigh frame.
- 12 Select the thickness of the condylar cushions (I) and secure them to the insides of the hinge with the Velcro.

### Fitting the tibial edge cushion (optional)

- 13 Affix the **tibial edge cushion (F)** medially or laterally from the tibial edge in the lower leg frame to relieve the tibial edge. The second cushion may also be affixed if desired.

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